


CONTENTS: 1x Oral Rehydration flow control set
with 1000 ml Supply Bag

INSTRUCTIONS FOR USE

1. Open the sterile pack.
2. Ensure that the tube is clamped with the roller clamp.
3. Pour the Oral Rehydration Solution (ORS) into the supply bag (FIG. 1)
4. Connect the supply bag to the giving set (FIG. 2).
5. Hang the supply bag so that the black marker is at the level of the patient (about 1.3m higher than the patient).
6. Adjust the flow rate on the flow controller to the prescribed value (25ml/h to 250ml/h) (FIG.4).
7. Flush the giving set with the Rehydration fluid.
8. Attach the set to the inserted gastric tube.
9. Unclamp the tube and monitor the flow of fluid to the patient.

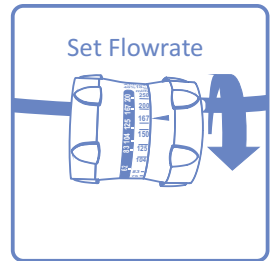

CAUTIONS & WARNINGS

1. Product is single use only.
2. Use clean, safe, water that is fit for human consumption.
3. Do not use for administering milk-based feeds or other enteral feeds.
4. Not for intravenous (IV) use.
5. Replace the line within 48-72 hours.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.



Do not use if unit
package is opened or
damaged.




Single Use



If re-used the sterility
of the device is
compromised, which
will greatly increase infection risk



See Instructions
for Use

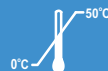

Avoid Extreme
Temperatures



Latex free



DEHP

MANUFACTURED IN SOUTH AFRICA



SINAPI biomedical, ARC Infruitec North Campus
Lelie Road, Stellenbosch, 7600, South Africa
Tel: +27 21 887 5260 Fax: +27 21 887 3059
email: sales@sinapibiomedical.com



Use By

